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Maahantuoja: IP-Agency Finland Oy

Arvo-rollator assembly instructions

1. Open your rollator and unfold the frame. The spring fastening piece on the right side should snap into place automatically. If not, check that rear wheels are extended all the way back and push the red fastening piece to its place.

2. Remove the black plugs from the rollators frame and insert the handles into place. Take the plastic levers and assemble them as shown below. Adjust the handles to desired height and angle. Ensure the handles are properly fastened to prevent injury caused by loose handles.

3. Insert transparent plastic tray into the four holes on the cushion.
4. Hang the basket on the hooks. Adjust its angle to facilitate loading.
5. Check and adjust the brakes to ensure they operate properly.

WARNING

Read all instructions before assembly or use. Before using the rollator, ensure your weight is within the unit's capacity. The rollator requires little assembly and the handles can be easily adjusted.

Parts to be assembled:

- 2 handlebars with loop lock brakes
- 2 plastic levers
- 1 basket
- 1 transparent plastic tray

Specifications:

- Steel frame
- Foldable
- Adjustable handles
- Locking handbrakes (press down on the brake lever)
- 8" solid rubber tires
- User max. weight: 110kg
- Weight: 13kg
- Height of the bench: 60cm
- Dimensions: 59 x 78-96cm
- CE-mark

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Importör: IP-Agency Finland Oy
- Importeur: IP-Agency Finland Oy